## Instructions for Use

# Zimmer® Collagen Plug Zimmer® Collagen Tape Zimmer® Collagen Patch Absorbable Wound Dressings for Dental Surgery

#### Distributed by:

Zimmer Biomet Dental
4555 Riverside Drive
Palm Beach Gardens

FL 33410 USA

+1 561.776.6700

zimmerbiometdental.com

## ENGLISH

#### Instructions for Use

#### Zimmer® Collagen Plug

#### Zimmer® Collagen Tape

#### Zimmer® Collagen Patch

#### Absorbable Wound Dressings for Dental Surgery

Before using this product, the surgeon/practitioner should carefully study the indications, contraindications, recommendations, warnings and instructions and fully comply with them. The manufacturer, the importer and the distributor of these products are not liable for complications, other negative effects or damages that might occur for reasons such as incorrect indications or surgical technique, unsuitable choice of material or handling thereof, unsuitable use or handling of the instruments, asepsis and so on. The surgeon/practitioner is responsible for any such complications or other consequences. It is also the surgeon/practitioner’s responsibility to properly instruct and inform the patient on the functions, handling and necessary care of the product and on all known product risks.

#### Description

Zimmer Collagen Plug, Tape and Patch are absorbent collagen matrices of porcine origin intended for the management of oral wounds and sores. Zimmer Collagen Plug, Tape and Patch are supplied in the form of a porous collagen matrix. Zimmer Collagen Plug, Tape and Patch are applied directly to the wound and protect the wound bed and delicate new tissue.

#### Indications

Zimmer Collagen Plug, Tape and Patch are indicated for the management of oral wounds and sores, including:

- Denture sores
- Oral ulcers (non-infected or viral)
- Periodontal surgical wounds
- Suture sites
- Burns
- Extraction sites
- Surgical wounds
- Traumatic wounds

#### Contraindications

Zimmer Collagen Plug, Tape and Patch are not indicated for patients with a known allergy to porcine-derived collagen materials. Large tissue defects should not be filled with several layers of Zimmer Collagen Plug, Tape and Patch products.

#### Warnings

Clinicians should use care in screening their patients for any known allergies to collagen or porcine-derived products.

#### Precautions

Zimmer Collagen Plug, Tape and Patch should not be used on infected or contaminated wounds.

Discontinue use of Zimmer Collagen Plug, Tape and Patch and notify your doctor if excessive redness, pain, swelling or blistering occurs.

Zimmer Collagen Plug, Tape and Patch have not been evaluated in pregnant women or children.

Zimmer Collagen Plug, Tape and Patch cannot be resterilized. Open, unused Zimmer Collagen Plug, Tape and Patch must be discarded. In vivo stability may be adversely affected if resterilized. Cross contamination and infection may occur if reused.

#### Adverse Reactions

Hypersensitivity reactions have been noted with the use of other products containing collagen.

#### Sterility

The product is supplied sterile, non-pyrogenic, and for single use only.

#### Administration

The wound should be rinsed and excess fluid removed. Zimmer Collagen Plug, Tape and Patch should be applied over the wound and held in place until adherence has been achieved. At the end of the procedure, Zimmer Collagen Plug, Tape and Patch can be removed, replaced or left in situ. Zimmer Collagen Plug, Tape and Patch may be replaced if they fall off the wound prematurely. Replacement after premature removal of Zimmer Collagen Plug, Tape and Patch is at the clinician’s discretion.

#### Storage

The product should be stored at room temperature.

#### How supplied

Individually packaged, supplied in boxes of 10, available in a variety of shapes and sizes:

Plug - 10mm x 20mm; Tape – 2.5cm x 7.5cm; Patch – 2cm x 4cm

#### Caution

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician or dentist.

©2017 Zimmer Biomet. All rights reserved.

Zimmer is a registered trademark of Zimmer Biomet or its affiliates.

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