Azure Endosseous Dental Implant Accessories / Azure 牙齒骨內植入物附件(未滅菌)

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“牙王” 牙齒骨內植入物附件(未滅菌) Azure Endosseous Dental Implant Accessories (Non-sterile)

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English

Azure Endosseous Dental Implant Accessories (Non-sterile)

※Please carefully read instructions for use before use and follow the directions during use.

Product description

Azure endosseous dental implant accessories consist of implant surgical tool kit and tooth impression part kit. They are suitable for Azure C or E implant systems for implant surgery and used for implantation of artificial dental roots as well as impression taking for manufacture of crowns. The product is made from titanium alloy (Ti64Al4V), stainless steel (Trimrite, SUS316L), POM, PA11, PSU and other materials.

Indications

The product is accessories used in dental implant surgery and mainly used as auxiliary tools during dental implant surgery. Dentists and dental technicians use the accessories in the mouth of the patient for dental implant surgery, and make dental models in this process to determine the correct position of the implant so as to manufacture correct crowns in dental laboratory after impression taking. Users should be the dentists and dental technicians familiar with the implant product surgical procedures of our company.

Precautions

1.The titanium alloy parts in the accessories shall be subjected to

sterilization process in a high temperature steam sterilizer before use. The sterilization conditions are as follows:

Gravity displacement sterilizer

Min. sterilization temperature (°C)	Min. exposure time required (min)	Min. drying time required (min)
121	30	45
132	15	45
135	10	30

Vacuum sterilizer

Min. sterilization temperature (°C)	Min. exposure time required (min)	Min. drying time required (min)
132	4	30
135	3	16

- The product should be stored in ventilated and cool places in rooms, and kept away from direct sunlight.
- In case of product damage and part missing before use, do not use, and contact our company as soon as possible for replacement and maintenance.
- For details on usage of accessory products, please consult the surgical instructions or contact technical sales.

Product specifications

As listed in product specification table.

Azure™

Manufacturer:
INTAI TECHNOLOGY CORP.
Manufacturing Address:
9, Jingke Rd., Nantun
District, Taichung City, Taiwan,
40852, R.O.C
TEL: 886-4-3601-5336
FAX: 886-4-3601-9772
www.intai.com.tw

VIET03-01