 **Italmed-Ad S.r.l.**
Via Masaccio 32, Firenze

 **Biomet 3i, LLC**
dba ZimVie Dental
4555 Riverside Drive
Palm Beach Gardens, FL
33410, USA
+1-561-776-6700
<http://www.zimvie.com/en/dental.html>

MD

UDI 8013805745747ITD022IB153

REF BHA5

CE 0373

2°C  +25°C

ifu.biomet3i.com

BIOTIVITY™ HYALURONIC ACID

BIOTIVITY™ HYALURONIC ACID

hyaluronic acid sterile solution for dental use

Indications

Implant surgery

- a) as an adjuvant treatment in the regeneration of bone defects of the peri-implant alveolar ridge in association with bone substitutes and resorbable membranes.
- b) protection of the implant surface, implant site and sutures.

Treatment of peri-implantitis

As an adjuvant treatment in bone regeneration processes also in association with bone substitutes and with resorbable membranes.

Extractive surgery

As an adjuvant treatment in the bone regeneration of the alveolus and prevention of post-extraction alveolitis.

Composition

Sodium hyaluronate, monobasic potassium phosphate, dibasic sodium phosphate, water for injections

Aspirate the desired amount of solution from the vial with the sterile syringe

The fluid gel that is not used can be stored in the vial, away from light and refrigerated at 2-4° C, for up to 5 weeks.

Implant Surgery

a) as adjuvant treatment for regeneration of periimplant bone defects, mix the product with the bone particulate in order to obtain a smooth paste, remove the excess fluid gel with sterile gauze, then apply the product into the bone defect, modelling it with a sterile instrument.

b) to protect the implant and the area of the implant, apply the fluid gel bending the needle with the blunt tip to mimic a periodontal probe.

Place the blunt tip of the needle near the base of the area of the implant and inject the product until the fluid gel flows from the top edge. Remove the needle from the area.

Last revision: June 2024

In case of any adverse event or malfunction please notify as soon as possible:

EMEA: emeacomplaints@zimvie.com

US & Canada: DentalComplaints@zimvie.com

International: DentalInternationalComplaints@zimvie.com

Packaging Contents

- 1 unit multi-dose bottle 5 ml sterile solution
- 5 units sterile disposable syringes 2.5 ml

Warnings

- » Medical Device for professional use only
- » Sterile product
- » Multi-dose bottle for at least 5 applications
- » Do not reuse the needles and syringe: dispose them immediately after use
- » Not for injection
- » Keep away from heat sources and out of reach of children
- » Do not use after the expiration date
- » Store at a temperature not exceeding 25 ° C.

Dispense several drops of the product directly on the implant immediately prior to the insertion into the area so that the entire surface of the implant is wet.

Treatment of peri-implantitis

Mix the product with bone particulate in order to obtain a smooth paste, remove the excess fluid gel with sterile gauze then apply the product into the bone defect, modelling it with a sterile instrument.

Extractive Surgery

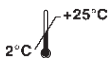
In particular for delayed implant placement mix the product with the bone particulate in order to obtain a smooth paste, remove the excess fluid gel with sterile gauze, then apply the product into the bone defect, modelling it with a special sterile instrument.

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