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ENGLISH	Ti-System
	Reprocessing Instruction acc. to DIN EN ISO 17664
Procedure:	Mechanical reprocessing with manual pre-cleaning
Products:	All reusable instruments of the Ti-System
Note:	Frequent reprocessing has little effect on the instruments of the Ti-System. The result of the function testing performed as part of the maintenance and control determines the service life of the product.
Instructions:	
Storage and transport:	No special requirements. Reprocessing should be done as soon as possible [after use] to avoid damage to the instruments and contamination of the environment.
Place of use:	Remove gross contamination from the instruments directly after use, and immerse them into cold water.
Cleaning preparation:	Disassemble the instruments where possible or open them, respectively, and rinse them under running water until all visible contamination is removed. If necessary, use a soft brush or, for cavities, a water jet.
Machine cleaning:	Place the instruments into the machine so that the water can flow out of the channels and openings. - 4 minutes pre-cleaning with cold water - 6 minutes cleaning at 55 °C (131 °F), with addition of 0.5% alkaline cleaner - 3 minutes neutralization at a temperature below 40 °C (104 °F) - 2 minutes rinsing with water at a temperature below 40 °C (104 °F) Apart from that, the device-specific instructions are to be followed.
Disinfection:	Perform thermal machine disinfection in compliance with all national requirements regarding the A0 value (see EN 15883).

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Drying:	According to the automatic drying program of the cleaning and disinfecting machine. If necessary, dry manually with a lint-free cloth. Dry cavities with sterile compressed air.
Inspection and functional test:	All instruments: Perform visual inspection for damage and wear. In case of demountable instruments, check the connections with the associated components.
Packaging:	Package the products into suitable packaging material. The bag must be large enough for tension-free sealing.
Sterilisation:	Sterilisation of the products is done using a fractionated pre-vacuum process (according to ISO 13060 / ISO 17665) and in compliance with the respective national requirements. Heat up to a sterilisation temperature of at least 134 °C (274 °F). Retention time: 5 min. Drying time: at least 10 min.
Storage:	Store the sterile instruments in a clean, dry and dust-free place.
Additional Information:	The above instructions have been validated by the medical device manufacturer as suitable for the preparation of a medical device for reuse. It is incumbent upon the user to ensure that the reprocessing actually performed with the equipment, materials and staff at the reprocessing facility achieves the desired results. To this end, validation and routine monitoring of the process and the equipment used are usually required.
Manufacturer contact:	curasan AG Lindigstrasse 4 63801 Kleinstheim Germany Phone: +49 (0) 6027 / 40900-0 Fax: +49 (0) 6027 / 40900-29 Email: info@curasan.de www.curasan.de
	Date of information April 1, 2015

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