CONTENT

BG (Български) Ръководство за употреба GenTek™ Шестостенна отвертка	2
CS (Česky) Návod k použití GenTek™ hexalobulární šroubovák	4
DA (DANISH) Brugervejledning GenTek™ Hexalobular Screwdriver	6
DE (Deutsch) Gebrauchsanweisung GenTek™ Sechsrund-Schraubendreher	8
EL (Ελληνικά) Εγχειρίδιο οδηγιών GenTek™ Εξάλοβο κατσαβίδι	10
EN (English) Instruction Manual GenTek™ Hexalobular Screwdriver	12
ES (Español) Manual de instrucciones GenTek™ Destornillador Hexalobular	14
ET (Eesti) Kasutusjuhend GenTek™-i kuusnurkne kruvikeeraja	16
FI (Suomi) Käyttöopas GenTek™ kuusikulmainen ruuvimeisseli	18
FR (Français) Mode d'emploi Tournevis hexalobé GenTek™	20
HR (Hrvatski) Priručnik GenTek™ Heksalobularni Odvijač	22
HU (Magyar) Kezelési útmutató GenTek™ torx csavarhúzó	24
IT (Italiano) Istruzioni d'uso Cacciavite esalobato GenTek™	26
LT (Lietuvių k) Instrukcijų vadovas GenTek™ šešiaskiltis atsuktuvus	28
LV (Latviski) Lietošanas rokasgrāmata GenTek™ sešstūrveida skrūvgriezis	30
NL (Nederlands) Instructiehandboek Hexalobulaire GenTek™ schroevendraaier	32
PL (Polski) Instrukcja obsługi Wkrętek torx GenTek™	34
PT (Português) Manual de Instruções GenTek™ Chave de parafusos Hexalobular	36
RO (Română) Manual de instrucțiuni Șrubelni și hexalobată GenTek™	38
SK (Slovak/Slovakian) Návod na použitie Skrutkovač na skrutky s hexalobulárnou drážkou GenTek™	40
SL (Slovenščina) Navodila za uporabo Heksalobularni izvijač GenTek™	42
SV (Svenska) Instruktionsmanual GenTek™ Hexalobular Skruvmejsel	44

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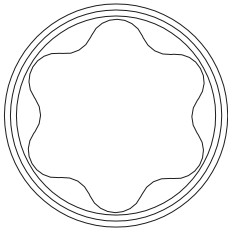
Instruction Manual GenTek™ Hexalobular Screwdriver

Version: 2/05.2021
Compiled at: 19.05.2021
Effective date: 19.05.2021
Replaces version: 1/04.2021

Indications for Use

The GenTek™ Hexalobular Screwdriver is used to tighten / loosen GenTek™ Hexalobular (laboratory try-in or final prosthetic) Screws to enable assembly / disassembly of GenTek™ single-unit (engaging) and multi-unit (non-engaging) restorative systems containing the hexalobular screw to corresponding Zimmer Biomet Dental Implant Systems.

Hexalobular Drive

Device description

The GenTek™ Hexalobular Screwdriver is used to drive the GenTek™ Hexalobular screw through a restorative screw access channel containing an angulation of up to 25°.

The screwdriver is designed with an ISO-latch connection allowing for use with Zimmer Biomet Low Torque Indicating Ratchet Wrench Standard ISO 1797 Adapter* [Item #: C9980]. The Zimmer Biomet Low Torque Indicating Ratchet Wrench Standard ISO 1797 Adapter [Item #: C9980] is intended to be used with the Zimmer Biomet Low Torque Indicating Ratchet Wrench (L-TIRW).

The screwdriver is an individually packaged reusable component and is available in various lengths. The screwdriver is provided non-sterile and must be cleaned and sterilized prior to use in a patient's mouth.

Materials

GenTek™ Hexalobular Screwdriver are manufactured from medical grade stainless steel.

Procedure


× Handling

1. Connect the ISO-latch portion of the driver into Low Torque Indicating Ratchet Wrench Standard ISO 1797 Adapter [Item #: C9980]
2. Engage the screwdriver to the corresponding GenTek™ Hexalobular Screw to tighten or loosen

Refer to the below table for placement torque recommendations for GenTek™ Hexalobular (laboratory try-in or final prosthetic) Screws:

Connection*	Angulation Range	Laboratory Try-in Screw	Final Prosthetic Screw Torque	GenTek™ Hexalobular Driver*
Certain®	0–25 degrees	Hand-tighten**	20 Ncm	ZFX02HLD21, ZFX02HLD28, ZFX02HLD36
TSV® / Trabecular Metal™	0–25 degrees	Hand-tighten**	30 Ncm	ZFX02HLD21, ZFX02HLD28, ZFX02HLD36
Eztetic®	0–25 degrees	Hand-tighten**	30 Ncm	ZFX02HLD21, ZFX02HLD28, ZFX02HLD36
External Hex	0–25 degrees	Hand-tighten**	32 Ncm	ZFX02HLD21, ZFX02HLD28, ZFX02HLD36

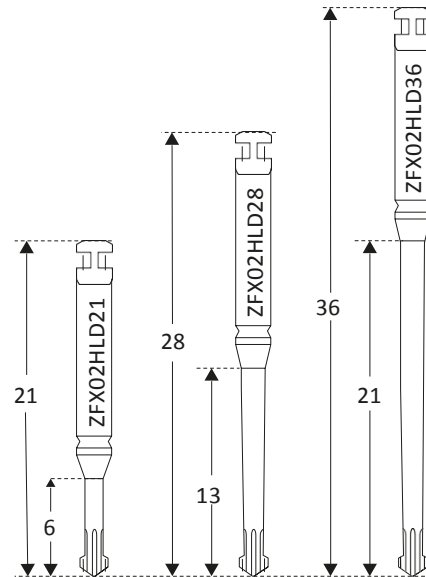*Applicable to single-unit and multi-unit for each connection
**Minimum of 5 Ncm; Maximum of 15 Ncm

× Cleaning and Sterilization

The GenTek™ Hexalobular Screwdriver is non-sterile. Prior to use, the driver must first be cleaned and sterilized. The driver can be used up to 15 cleaning / sterilization cycles.

Cleaning

1. Rinse the driver under running cold tap water for 2 minutes.
2. Hand wash instruments in a bath with demineralized water for 5 minutes. Remove debris with a soft brush. Cavities should be rinsed with a syringe. This step should be repeated until all debris is removed.
3. Clean the instrument in an ultrasonic bath using a mildly alkaline cleaner (1.5%) at a frequency of at least 40 kHz for 20 minutes.
4. Rinse thoroughly with demineralized water for 1 min.

Sterilization

The recommended sterilization procedure is full cycle pre-vacuum steam sterilization at a temperature of 134 °C (273.2 °F) for 3 minutes, 4 pulse minimum or full cycle gravity steam sterilization at 134 °C (273.2 °F) for 3 minutes minimum. The products are ready for use 5 minutes after completion of the autoclaving process.

Note: Follow the loading instructions issued by the sterilizer manufacturer.

Storage and Handling

The GenTek™ Hexalobular Screwdriver should be stored at room temperature.

Contraindications

None

Known Side Effects

None

Instruction Manual

GenTek™

Hexalobular Screwdriver

Version: 2/05. 2021
Compiled at: 19.05. 2021
Effective date: 19.05. 2021
Replaces version: 1/04. 2021

Warnings

- ✗ The GenTek™ Hexalobular Screwdriver must only be used and handled by dental professionals who have approved training in dentistry.
- ✗ GenTek™ Hexalobular Screwdriver may only be used for GenTek™ Hexalobular Screws.
- ✗ The GenTek™ Hexalobular Screwdrivers are small and, therefore, they must be handled with caution to avoid swallowing or inhaling by the patient.
- ✗ The use of a screw torque higher than the recommended value may result in damage to the GenTek™ Hexalobular Screwdriver, GenTek™ TiBase, screws, custom abutment, and/or the implant.
- ✗ The screwdriver must be in good condition. A worn screwdriver can lead to over-rotation of the hexalobe in the screw head which can result in the screw not being fully tightened and / or not being able to be removed.

Explanation of symbols

Non-sterile

Do not use if package is damaged

Manufacturer Information: Address of legal manufacturer facility

Unique Device Identifier

LOT-Number, Symbol followed by Production Date of the lot.

Medical Device

Reference Number, Symbol followed by Manufacturer's Item Number

Quantity/Unit of package content

Consult instructions for use
www.zfx-dental.com

Consult instructions for use for mating hexalobular screws and angulated screw channel TiBase abutments

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Zfx GmbH